

How to put on a mask

1

Sanitize your hands.



2

Pick up mask using ear loops.



3

Expand the mask. (Colored side faces out.)



4

Place mask on your face by securing loops around your ears.



5

Adjust mask to cover your chin and pinch metal nose piece to give a secure fit.



How to remove a mask

1

Sanitize your hands.



2

Handle only by ear loops to remove mask.



3

Dispose of mask immediately.



4

Sanitize your hands.



UFHealth
UNIVERSITY OF FLORIDA HEALTH